

Tips For Use

- Under certain conditions, condensation may form on the inside of the ShowerSafe due to sweating during showering or bathing. Lightly wrap affected limb with a thin towel to absorb moisture if desired.
- Turn ShowerSafe inside out after use to allow the inside to dry thoroughly between uses.
- If the ShowerSafe is punctured or torn, repair with waterproof tape (duct tape, waterproof first aid tape, etc.)
- Non-skid tape may be applied to the ShowerSafe cover to help prevent slipping when wet.
- Carefully trim the rubber WrapAround strap if it is too long.
- Occasionally wash the inside of the ShowerSafe cover with warm water and soap to prevent odor build up.
- The ShowerSafe bag may be rolled up from the closed end, and taped, if the cover is longer than needed.